

Equine Sample Collection Sheet

Hair Collection

Step 1: Fill Out the Following Information (One form per tested horse)

Submitter's Name: _____ Date of Collection: _____
Horse's Name: _____ Gender: Male Female
Breed: _____ Color / Markings: _____

Step 2: Collect the Sample

Pull – **DO NOT CUT** – hair samples from the mane above the withers. Pull 12-25 hairs by wrapping the hair around your finger close to the skin and pulling quickly and evenly. The DNA is contained in the root bulb (follicle) – it is important that this part of the hair remain intact. Inspect the hairs to see that they have not broken off before the root. For young foals, the hair from the mane tends to be finer with a smaller follicle. Pull hair from the tail of the foal instead.


Step 3: Attach Hair Sample to this Form

Place the hairs in a straight line across the middle of this page in the **Hair Sample Section** below. Gather the hair, with the roots to the left, as indicated. Tape the center of the hair shafts with CLEAR tape to ensure the samples stay in place.

Hair Sample Section (Attach hairs here, as shown in example. DO NOT CUT HAIR)

Place Hair
Roots Here
(No Tape)

15-25 Hairs
Tape Center of
Hair Shafts Here

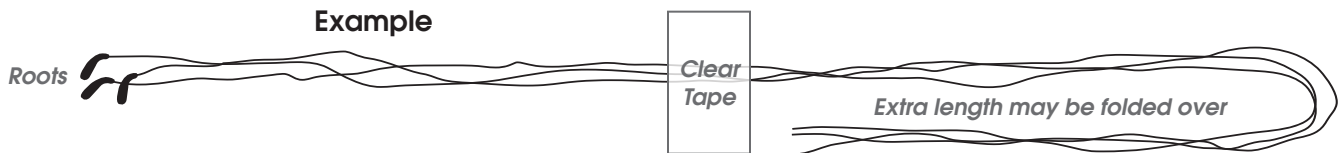

Step 4: Mail to DDC Veterinary

Complete the DNA Testing Sample Submission Form located online at www.vetdnacenter.com/files/dna-test-orderform.pdf. Please note that an **email address is required for receiving results**. Then, fold this form at the dashed lines and place into a mailing envelope with the DNA Testing Sample Submission Form and payment (if applicable). Send to:

DDC Veterinary
One DDC Way
Fairfield, OH 45014 USA

Questions? Contact DDC Veterinary at 1-800-625-0874 or contact@vetdnacenter.com